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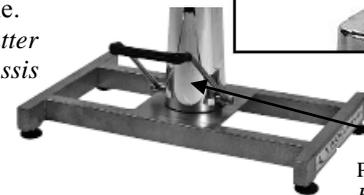
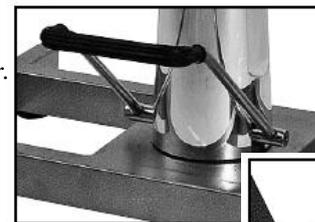
**Characteristics**

1-year guarantee  
Specially designed for small to medium sized dogs  
Variable height - 90 < 104 cm with the wood design table top  
- 94 < 108 cm with the polyethylene table top  
1 integrated storage basket  
1 adjustable bracket + 2 straps  
Hydraulic jack with foot control

Position finale du vérin  
Final position of the cylinder

**Cleaning and Maintenance**

The table doesn't require any maintenance: wash with tepid water, use a damp cloth and a soft washing liquid.  
Do not use inappropriate solvents and aggressive liquids (e.g. cream, greases, etc.).

**Instruction for use**

- Press in a brief repeatedly pedal jack to adjust the height of the table according to the needs.  
- To lock the table top (avoid it running), raise the pedal fully upwardly by pressing it  
- To lower the height of the table, press up on the pedal

**Guarantee**

The table is guaranteed for a period of 12 months from the date of purchase, the guarantee terminates at this time even if the table has not been used. It is therefore advisable to keep the proof of purchase (receipt and original packaging). The guarantee is applicable for normal conditions of use and not intensive, with regular maintenance of all parts. The guarantee covers all mechanical manufacturing defects.  
The replacement or repairs of parts that are subject to normal wear or resulting from misuse are therefore excluded.  
Any repairs made during the guarantee period do not result in a prolongation of the latter. The guarantee is not valid if the table has been damaged as a result of negligence, inexperience, or during transport. The replacements or repairs covered by the guarantee are free of charge by sending the table directly, in its original packaging, with proof of purchase to:

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**VIVOG SAS**