Table hydraulique « tournante » STAR UNIVERSAL TH700
Rotating hydraulic table STAR UNIVERSAL TH700

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Characteristics

1-year guarantee
Specially designed for small to medium sized dogs and cats
Variable height - 78 < 92 cm
1 adjustable bracket + 1 strap
1 black non-slip mat, comfortable and easy to clean
Hydraulic jack with foot control and anti-rotation lock for the table top

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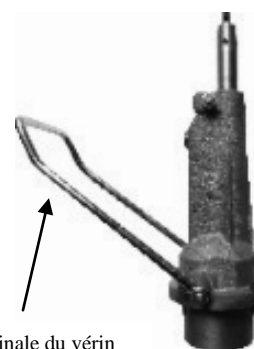
Instruction for use

- Press in a brief repeatedly pedal jack to adjust the eight of the table according to the needs.
- To lower the height of the table, press up on the pedal

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Slip the hydraulic pump enclosure (m) into hydraulic cylinder (o), fasten them together with screw (n)

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Put the diagonal brace base (s) on the top of hydraulic cylinder (o), use the spanner (j) to fasten them together.

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Use spanner to screw the washer (b) and the steel screw nut (c) together

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Put the 4 diagonal braces (r) in the four directions and fasten them with 4 washers (d) and 4 screws (e) with the spanner (j)

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Put the diagonal braces (r) under the table, align position, fastens them together one by one with 4 washer (f) and 4 screws (g) with the spanner.

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Cleaning and Maintenance

The table does not require any maintenance: wash with tepid water, use a damp cloth and a soft washing liquid.

Do not use inappropriate solvents and aggressive liquids (e.g. cream, greases, etc.).

Guarantee

The table is guaranteed for a period of 12 months from the date of purchase, the guarantee terminates at this time even if the table has not been used. It is therefore advisable to keep the proof of purchase (receipt and original packaging).

The guarantee is applicable for normal conditions of use and not intensive, with regular maintenance of all parts.

The guarantee covers all mechanical manufacturing defects.

The replacement or repairs of parts that are subject to normal wear or resulting from misuse are therefore excluded. Any repairs made during the guarantee period do not result in a prolongation of the latter.

The guarantee is not valid if the table has been damaged as a result of negligence, inexperience, or during transport.

The replacements or repairs covered by the guarantee are free of charge by sending the table directly, in its original packaging, with proof of purchase to:

VIVOG SAS

2 Rue de la Briaudière CS 4000

37515 BALLAN-MIRÉ CEDEX- FRANCE.

tel. +33 (0)2 47 73 38 38 - fax. +33 (0)2 47 73 38 48

email : infos@vivog.fr

Shipping costs are at the expense of the purchaser.